

CARE

Poultice & Wraps



Versatile multi-layered poultice for wound dressing. Can be used warm and wet, cool and wet, or dry.



EASY TO APPLY



WOUND CARE



LIGHT WEIGHT



Multi-purpose cohesive bandage wrap suitable for dressing wounds or providing support.



SELF ADHESIVE



WATER RESISTANT



LIGHT WEIGHT

Poultice

HOW TO USE

Cut dressing to size and apply to affected area with plastic backing away from skin using one of the following methods:

Warm and Wet: Allow boiled water to cool to 38°C, soak briefly, lightly squeeze and apply dressing.

Cool and Wet: Chill boiled water, soak briefly, lightly squeeze and apply dressing.

Dry: Apply dressing directly to affected area.

Hold dressing in place using a Plusvital Wrap cohesive bandage (avoid applying too tightly). Dressings should be changed at least every 12 hours. Consult a veterinary surgeon if condition has not improved in 72 hours.

WARNING: Consult a veterinary surgeon if Plusvital Poultice is to be used in conjunction with medical preparations.

AVAILABLE SIZES

SUITABLE FOR ALL HORSES

Sold in boxes of 10

20cm x 40cm

Wraps

HOW TO USE

Do not apply too tightly.

Apply the last turn without tension, and press firmly in place


Change regularly, at least every 24 hours.

WARNING: This product contains natural rubber which may cause allergic reactions.

AVAILABLE SIZES

SUITABLE FOR ALL HORSES

Sold in boxes of 18

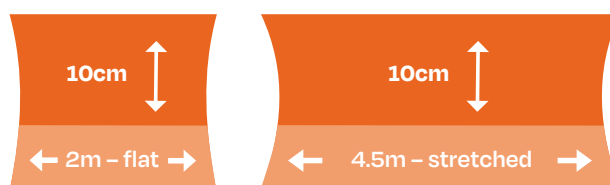

Available in a variety of colours.